

PREvention™ HLD



8 Minute High Level Disinfectant (HLD) for
Spa & Salon Tools, Powered by **ACCELERATED®**
HYDROGEN PEROXIDE Patented

Why Use a High Level Disinfectant?

Semi-critical instruments that come into contact with non-intact skin or mucous membranes increase the risk of infection. Therefore, they require a higher level of care when reprocessing, which a HLD can achieve.

For Use On:

- Nail Clippers & Nippers
- Cuticle Pushers*
- Drill Bits**
- Foot Files*
- Tweezers
- Comedone Extractors
- Microblading Handles

*Stainless Steel
** Excluding Carbide

Prevention™ HLD8 Quickly Disinfects Tools Without the Use of Harsh Chemicals



TOUGH ON GERMS

- HLD8 has a **rapid 8 minute contact time** for disinfecting tools and can be re-used for **up to 21 days** in a soaking tray

SAFER FOR YOU

- HLD8 is **non-toxic**, and **does not require special ventilation**
- HLD8 **will not leave behind toxic residues** that are harmful to clients & staff




SAFER FOR THE PLANET

- HLD8 has **biodegradable** properties which allows it to be poured down the drain **safely**
- The active ingredient in HLD8, hydrogen peroxide readily breaks down into **Oxygen and Water**

How to Use PREvention™ HLD8^β


1 CLEAN

- A) Clean all instruments using a detergent*
- B) Thoroughly rinse and dry instruments

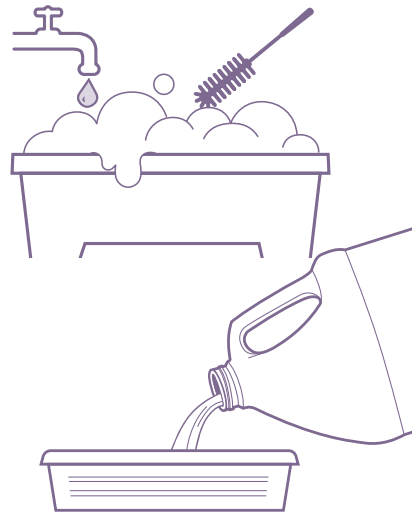

2 POUR

Pour fresh solution[‡] of Prevention™ HLD8 into clean soaking tray, no activation or dilution is required

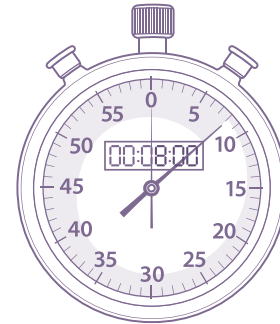

3 DISINFECT


- A) Fully immerse pre-cleaned dry instruments in Prevention™ HLD8 solution
- B) Wait **8 minutes**
- C) Rinse instrument with potable or sterile water and dry[†]
- D) Store in clean, closed container

How to Validate

Use Prevention™ HLD8 Test Strips to validate the required concentration of hydrogen peroxide during the re-use period up to 21 days


Record and Track the validation of your re-use solution using our helpful log charts available online

Dip
Hold
Match

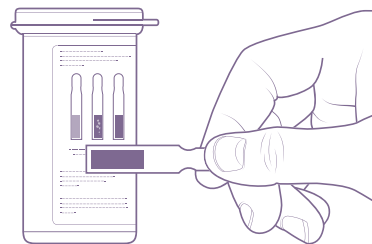

When to Discard Solution^{**}

- Up to 21 days
- If the solution is visibly soiled
- If indicator strip fails test

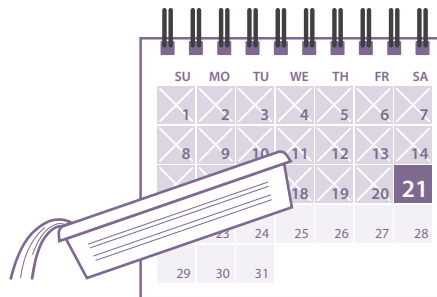

FDA Approved 8-Minute High Level Disinfectant

- Bactericidal, Virucidal, Fungicidal, Tuberculocidal
- GUDID#: 00068460505190
- Format Size: 1 Gallon

Did you Know?

The FDA is responsible for approving health care high level disinfectants in the US for use on semi-critical tools. The EPA is only responsible for reviewing surface disinfectants.

Test Strips for Prevention™ HLD8

- GUDID#: 10068460505135
- Format Size: 60 Test Strips
- Product Code: AHPHLD8
- Packaging Specifications: 1x60 Test Strips

Prevention™ HLD8 High Level Disinfectant Test Strips were designed to validate the Prevention™ HLD8 solution for up to 21 days. Test Strips are available for purchase through distribution.

For more information, visit:

ViroxProBeauty.com

2770 Coventry Road, Oakville, ON L6H 6R1 CANADA
Toll Free: 1-800-387-7578

^β Follow product label directions for use
[‡] Solution to be used at room temperature
^{*} Cleaning can be done either manually or mechanically.
^{**} For disposal of Prevention™ HLD8 follow local and state regulations.
[†] Refer to device manufacturer's recommendations for rinsing and drying instructions.

